

Tristel Shine

ENT decontamination with
Tristel Shine

A new washer disinfectant system for
non-lumened Nasendoscopes

Tristel

Content

- About Tristel
- Tristel Chemistry
- Tristel Sporocidal Wipes System for ENT
- The NEW Shine Washer Disinfectant for ENT

Tristel

About Tristel

- Leading supplier of high-level disinfectants for instruments to UK hospitals
- Selling into 60% of NHS Acute and Private hospitals
- Tristel products have been available for over 12 Years

Tristel

Chemistry

- Tristel products use unique, patented chlorine dioxide chemistry
- Rapidly sporicidal
- Chlorine dioxide is safe for patients, users & the environment

Tristel




Disinfection for instruments

Tristel Sporicidal Wipes System for ENT



Tristel

Tristel Wipes System

Stage 1	Stage 2	Stage 3
		
<p>Pre-Clean Wipes for removal of soils prior to disinfection</p>	<p>Sporicidal Wipes and activator foam pump for disinfection of the equipment. 30 second contact time</p>	<p>Rinse wipes for the removal of disinfection residues</p>

Tristel

Traceability System



The Tristel Quality Audit Trail Book allows the user to record the complete process of instrument decontamination


Tristel

Tristel Shine – Why?

- To comply with the Department of Health requirement for an automated decontamination process
- To reduce the risk of cross patient infection
- To comply with standards
HTM 2030
ISO 15883

Tristel

Shine – washer disinfecter

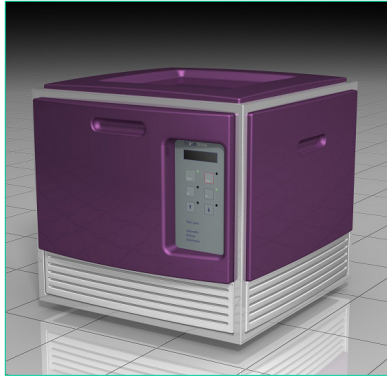


- Easy to use
- Pre programmed cycle
- One button operation
- Bench top or free-standing with a small foot print
- Compliant to HTM2030 and ISO15883/4
- Short 20 minute cycle (dependent upon correct services)

Tristel

Shine – washer disinfecter continued

- Printer supplied for traceability
- Compatible with RO water
- Automatic thermal self-disinfect cycles

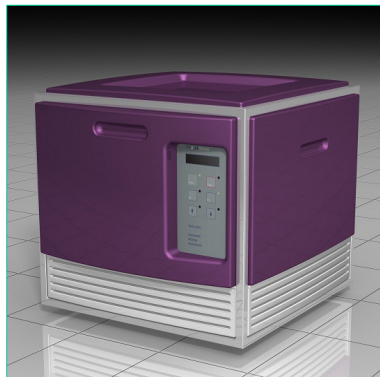

Tristel

Decontamination

- Up to four scopes per cycle



Tristel

Single-Use Solutions

- Tristel enzymatic detergent for pre-wash cycle
- Single-Use Tristel chlorine dioxide chemistry



1 x 5 Litres Detergent



1 x 5 Litres Base Solution



1 x 5 Litres Activator Solution

Tristel

Conclusion

The Shine ENT washer disinfecter and the Tristel Wipe System combined with the unique Tristel chemistry offers the Healthcare professional a,

- **Cost effective**
- **Efficient**
- **Easy to use**
- **Safe method,**

of decontaminating Nasendoscopes and non-lumened instruments

Tristel

Contacts

Tristel Head Office	Rachel Audus	01638 721500
South-West & South-Wales	Steve Wise	07824 453426
London & South East	Danielle Nugent	07887 992778
Midlands	Dave Herd	07900 058211
North	Richard Fish	07500 010300
Scotland & North East England	Lesley Murray	07900 058210
East Anglia	Kevin Menhinick	07836 724388

Tristel