

How to Use Tristel Life Foam

For all applications and uses:

- **Do not use the Tristel Life Foam if the bottle has been damaged or tampered with**
- **Before disinfection, use paper towel to remove leftovers of the gel from the probe.**



Step 1

Disinfect hands immediately before and after each patient contact.

Step 2

Put on gloves.

Step 3

Remove the transport locks on the Tristel Life Foam that stop the pump being depressed in transport. It is recommended that these are retained and replaced after using the Tristel Life Foam.

Step 4


If the Tristel Life Foam is being used for the first time, depress the pump two to four times to prime the foamer. When primed, depress the pump once to dispense one 0.8 ml aliquot of chlorine dioxide foam onto the probe or a wipe.

Step 5

Take a paper towel or a wipe (Tristel Multi Wipes are recommended) and disperse the Tristel Life Foam over the ultrasound probe. Ensure all surface has been covered with foam. Leave for 30 seconds and if necessary remove leftovers of the foam using a clean paper towel or a wipe.

Step 6

Discard the paper towel/wipe used to waste. Do not re-use.

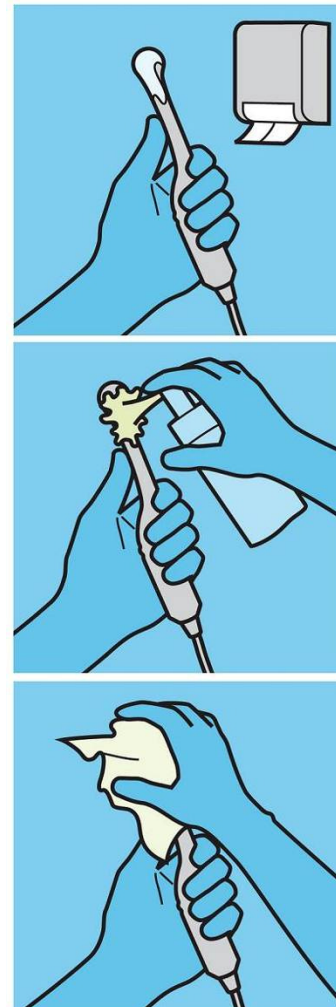

Manufactured by:

Tristel Solutions Limited

Lynx Business Park, Fordham Road, Snailwell, CB8 7NY, United Kingdom

Tel: + 44 (0) 1638 721500 Fax: +44 (0) 1638 721911 Email: mail@tristel.com