


How to Use Tristel Life Wipes System

For all applications and uses:

- **Do not use if the wipe sachet and/or foam bottle have been damaged**
- **Before disinfection, use paper towel to remove leftovers of the gel from the probe.**

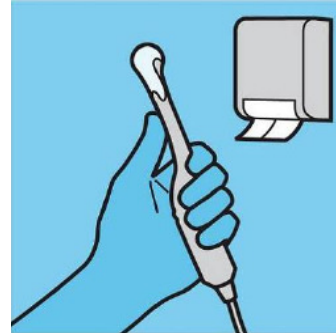

Using the Tristel Life Sporidical Wipe

Step 1

Put on gloves.

Step 2

To dispense a Sporidical wipe, take one sachet, tear and remove the wipe.

Step 3

Unfold the wipe and lay out on the palm of your hand.

Step 4


Take the lid off the foam bottle. Note that the foam bottle label is identified as **Tristel Life Activator Foam**.

If the foam bottle is being used for the first time, depress the pump two to four times to prime the foamer. The first output from the foam bottle can be left on the wipe, to be followed by two complete pumps. The foam bottle is now ready for subsequent use.


For all subsequent wipes, pump two measures of Tristel Activator Foam onto the wipe.

Step 5

Scrunch the wipe for at least 15 seconds to ensure it is covered with the Activator Foam. The foam has a yellow colour. The wipe is white.

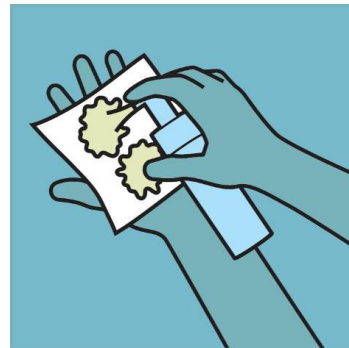

- Remember, activate the wipe as soon as you have removed it from the sachet and use it immediately. An activated wipe will have a faint odour of ClO_2 .

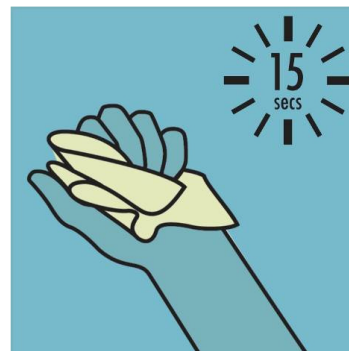

Manufactured by:

Tristel Solutions Limited

Lynx Business Park, Fordham Road, Snailwell, CB8 7NY, United Kingdom

Tel: + 44 (0) 1638 721500 Fax: +44 (0) 1638 721911 Email: mail@tristel.com

Using the Tristel Life Sporicidal Wipe (...continued)

Step 6

Wipe the surface of the transducer until it has been fully covered with chlorine dioxide. All areas of the surface must come into contact with the wipe at least once.

Step 7

Once the entire surface of the transducer has been wiped and covered with chlorine-dioxide, wait 30 seconds.

Step 8

Discard used Tristel Life Sporicidal Wipe to clinical waste.

- Rinse the disinfected ultrasound probe using Tristel Life Rinse Wipe



Using the Tristel Life Rinse Wipe

The Tristel Life Rinse Wipe contains 5ml sterile water.

Step 1

To dispense a wipe, take one sachet, tear and remove one wipe.

Step 2

Unfold the wipe and lay out on the palm of your hand.


Step 3

Thoroughly wipe the ultrasound probe that has been disinfected.

Step 4

Discard the wipe and gloves to clinical waste.

- Record the details of the decontamination procedure in Tristel Audit Trail Record Book

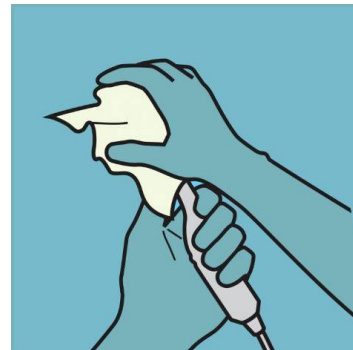

Rinse the instrument

Manufactured by:

Tristel Solutions Limited

Lynx Business Park, Fordham Road, Snailwell, CB8 7NY, United Kingdom

Tel: + 44 (0) 1638 721500 Fax: +44 (0) 1638 721911 Email: mail@tristel.com