


Tristel

Solo

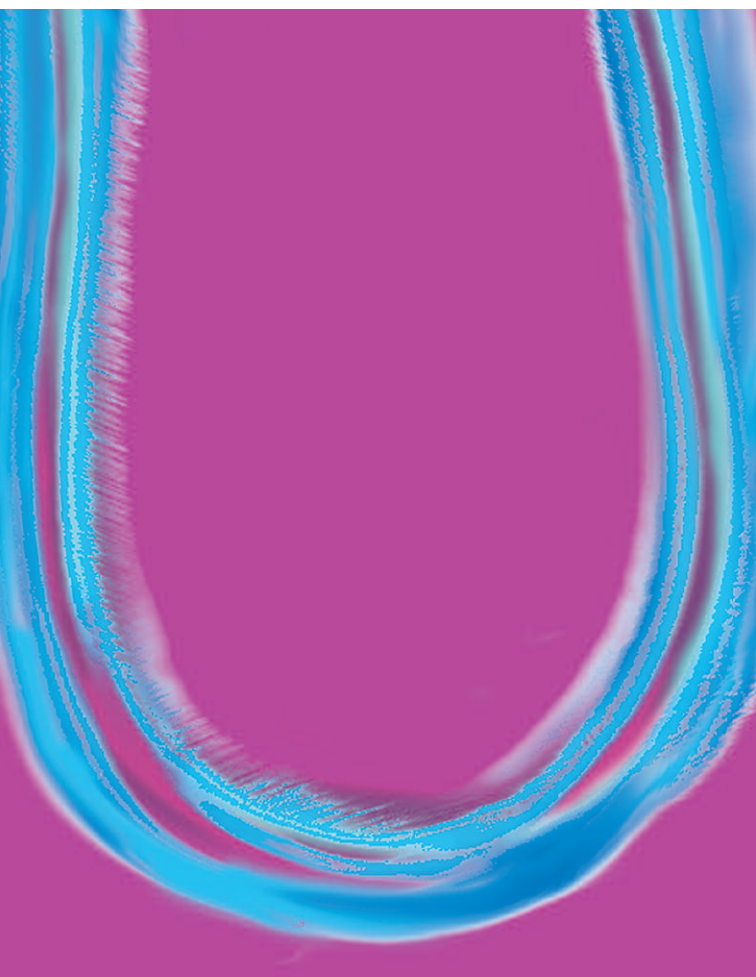

Cleaning and
disinfecting foam for
**non-invasive
ultrasound transducers**

User Guide

TRISTEL SOLO FOR ULTRASOUND

Tristel Solo for Ultrasound combines cleaning and disinfection in one simple process for the decontamination of non-invasive, skin surface ultrasound probes and ultrasound equipment such as monitors, keyboards and probe holders.

Tristel Solo Foam makes it possible to meet infection control guidelines for ultrasound equipment such as those set out in the paper entitled "An investigation of the microbiological contamination of ultrasound equipment" published in The British Journal of Infection Control, August 2006, Volume 7, number 4.

APPLICATIONS AND USES

Cleaning and disinfection of non-invasive, skin surface ultrasound probes and ultrasound equipment such as monitors, keyboards and probe holders.

BIOCIDAL PERFORMANCE

Tristel Solo for Ultrasound is bactericidal and virucidal. It has been extensively tested to validate its biocidal performance.

HOW TO USE

Step 1

Disinfect hands immediately before and after each patient contact.

Step 2

Remove lid from Tristel Solo bottle.

Step 3

If the Solo Foam is being used for the first time, depress the pump two to four times to prime the Foamer. When using the large 200ml Solo Foamer, depress the pump once to dispense one 0.75ml aliquot of Solo Foam onto the surface to be cleaned and disinfected. When using the small 50ml Solo Foamer, depress the pump twice to dispense 0.8 aliquot of Solo Foam onto the surface.

Step 4

Use a paper towel or a wipe to disperse the Solo Foam over the surface to be cleaned and disinfected.

Step 5

Discard the paper towel/wipe used to clinical waste. Do not re-use.

Step 6

Leave the surface to dry.

THE TRISTEL SOLO CHEMISTRY

Tristel Solo for Ultrasound utilises a polymeric biguanide combined with a highly effective surfactant system. Tristel Solo not only cleans the instrument surface but also disinfects it, being effective against *MRSA*, *E. Coli*, *Salmonella* and *Norovirus*.

CONTACT DETAILS

Tristel Solutions Limited
Cams, CB8 7NY, UK

Telephone: +44 (0)1638 721500

Website: www.tristel.com

