

Tristel Jet for surfaces

The Tristel Jet produces a high-level disinfecting and sporicidal gel for surface disinfection. When applied to a surface it can kill *Clostridium difficile* spores, MRSA and other pathogenic organisms in just 30 seconds. It is easy and ready to use and non-hazardous due to the non-aerosol technology.

According to The National Patient Safety Agency Healthcare Cleaning Manual June 2009, Infection Control Overview, “The essence of good cleaning is that things not only look clean afterwards, but that they are clean. All users of healthcare premises have a right to assume that the environment is one where infection hazards are adequately controlled”. Tristel Jet for surfaces will enable the user to comply with this.

FEATURES

- Gel formulation for accurate application
- Unique re-useable Trigger Head
- Sporicidal within 30 seconds
- Cleaning and disinfection combined
- Safe non-aerosol technology
- Available in a fragranced (faint smell of lemon) or an unfragranced format (faint smell of chlorine)

APPLICATIONS

- Commodes
- Mattresses
- Near patient surfaces
- Dressing trolleys
- General ward cleaning
- Patient hoists
- Other hard surfaces



NHS Supply Chain Codes:

FAL688 1 x Jet Trigger Head

FAL696 1 x refill pack fragranced

FAL697 1 x refill pack unfragranced

Tristel Codes:

JET/Head 1 x JET Trigger Head

JET/PACK/FRAG 1 x refill pack fragranced

JET/PACK/UNF 1 x refill pack unfragranced

Preparation Instructions



JET (trigger head)

REFILL PACK

JET PACK

Step 1

Remove both bottle caps from the Pack.

You will find one bottle closed by an insertion. The other bottle neck is open.

Step 2

Put the jet trigger head into the open bottle neck; then fasten the levers on each side of Jet trigger head to secure it to the bottle.

Step 3

Apply the gel onto a wipe or directly onto the surface. Wipe the surface ensuring it is completely covered. Leave to dry for 30 seconds.

When empty, dispose of the bottle pack and re-use the Jet trigger head for the next Jet Pack.