

# Tristel Clean™ for Stella

**Tristel Clean for Stella is a non-enzymatic detergent designed specifically for effective instrument cleaning prior to disinfection in Stella.**

A disinfection cycle is only effective when the medical device has undergone thorough cleaning.

Tristel Clean for Stella efficiently removes organic substances containing proteins, such as blood.

No intermediate rinsing is required if Tristel Fuse for Stella is used for high-level disinfection.

- Tristel Clean for Stella breaks down organic substances and proteins quickly and effectively
- Safe to use
- Two-year shelf life

Tristel Clean for Stella has an integrated 25ml dosing cap, meaning the correct dosage is easy to achieve for each application.




Tristel Clean for Stella is CE marked as a Class I Medical Device in accordance with the European Medical Device Regulation (EU) 2017/745 (MDR).

## How to use Tristel Clean for Stella

- Pre-clean the instrument to remove coarse dirt.




Prepare 25ml of Tristel Clean for Stella concentrate by squeezing the bottle until the marking on the cap is reached.

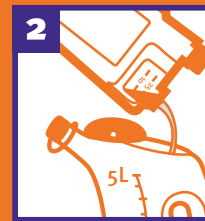


Pour the 25ml concentrate into a sink (for manual cleaning) or jug containing 5L of ambient water.

*Note: Do not use hot water.*

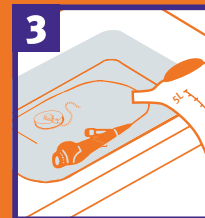

Pour diluted Tristel Clean for Stella into the sink or into the inner compartment of the base when prompted by Stella IQ.

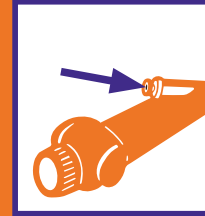

For manual cleaning: if the instrument has a channel, thoroughly brush it in the cleaning solution.



Ensure a contact time of 5 minutes.

- Remove the instrument from the cleaning solution and remove excess fluid.
- Insert the instrument into the Stella Disinfection System. Follow the instructions on the Stella display.

**Tristel™**  
WE HAVE CHEMISTRY.

**Manufactured in the United Kingdom by:**  
Tristel Solutions Limited, Lynx Business Park, Cambs, UK, CB8 7NY  
T +44 (0) 1638 721500 - E mail@tristel.com - W www.tristel.com

**Hong Kong & Taiwan:** 21st Floor, 168 Electric Road, Fortress Hill, Hong Kong  
T +852 2895 6968 - F +852 2869 4388 - E customerservice@tristel.com.hk

**New Zealand:** Tristel New Zealand Limited, 23 Birch Avenue, Judea, Tauranga P.O. Box 3110  
T +64 (0)7 5771560 - F +64 (0)7 5771567 - E info@tristel.co.nz

**Australia:** Tristel Pty Ltd, 40/328 Reserve Road, Cheltenham, VIC 3192  
T 1300 680 898 - F +61 (0)3 9533 6193 - E mail-au@tristel.com

Copyright © Tristel Solutions  
Mkt-Fac-786-1  
April 2019